fr

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en

## Instructions For Use Guarded Luer Connector

### Indications for use

The Guarded Luer Connector is a fluid transfer device used in the pharmacy to provide the fluid path for transferring large source container ingredients into a smaller container.

### Operating Instructions

#### \*Use Aseptic Technique\*

- Hold package firmly at tab and peel open while maintaining aseptic technique.
- Insert compatible luer connector from transfer container or fluid line into the desired luer connector on the device and rotate clockwise while pushing down.
- Remove the Guarded Luer Connector from the package with the transfer container or fluid line installed.
- Insert second compatible luer connector from second transfer container or fluid line into the remaining luer connector on the device and rotate clockwise while pushing down while maintaining aseptic technique.

**\*Verify the installation by rotating luer connectors until snug\***

### Contraindications

None when used as described in the device's Instructions for Use

#### Caution

- Do not re-sterilize
- Single use only. Do not reuse
- Clamp or close fluid line prior to connecting / disconnecting the Guarded Luer Connector
- Do not use if sterile barrier is broken

	Manufacturer
	Do not use if package is damaged
<b>LOT</b>	Batch code
<b>REF</b>	Catalogue number
<b>STERILEEO</b>	Sterilized using ethylene oxide
	Use-by-date

	Do not re-use
	Consult instructions for use
<b>Rx ONLY</b>	For sale by or on the order of a licensed health care practitioner
	Caution
	Non-pyrogenic

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International Medical Industries, Inc.